# **M** minniemobil



# Rollator Holder Instructions for Assembly and Use

#### **ROLLATOR HOLDER**

The minniemobil Rollator Holder allows you to transport your personal rollator when moving around with your minniemobil.

#### **COMPONENTS**

- 1 Rollator Holder platform
- 2 Bag Hooks
- 2 Stoppers
- 2 Screws



# **ASSEMBLY** (To be done by your Service Dealer)

- 1. Disassembly of the Warning Triangle Flap.
- 1.1. First unlock the Floor Plate of the minniemobil by removing the six hand screws that fasten it to the scooter (Fig. 1).
- 1.2. Then remove the Floor Plate with the help of the two handgrips on the rear of the Plate (Fig. 2).
- 1.3. Unscrew the two screws that fasten the triangle rods to the Scooter (Fig. 3).
- 1.4. Pull the Triangle Flap backwards to remove it (Fig. 4).





Fig. 1





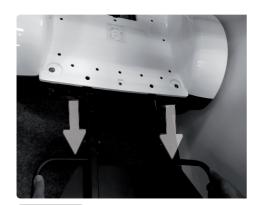


Fig.4

Fig. 2

#### 2. Assembly of the Rollator Holder Platform

- 2.1. Introduce the Rollator Holder Platform instead of the Triangle Flap (Fig. 5).
- 2.2. Fix it by placing the provided two screws with the accompanying washers.
- 2.3. Put the Floor Plate back into place.
- 2.4. Lock the Floor Plate with the six hand screws.

#### 3. Assembly of the Triangle Flap

- 3.1. Introduce the Triangle Flap into the end of the Rollator Holder (Fig. 6).
- 3.2. Fix it by screwing it to the Rollator Holder.



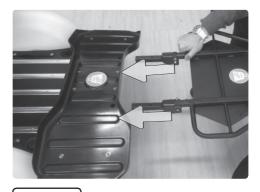


Fig. 5

Fig. 6

# 4. Disassembly of the Back Bar

- 4.1 The Back Bar is fixed to the scooter by two screws. Unscrew them (Fig. 7).
- 4.2. Then take the Back Bar out by pulling it back (Fig. 8).

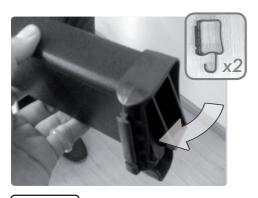




Fig. 8

#### 5. Attachment of the Bag Hooks

- 5.1. Slip the Bag Hooks on one of the Back Bar ends with the hook facing down (Fig. 9). The adaptor slot needs to face to the outside of the Back Bar.
- 5.2. Slip the Bag Hooks over the Back Bar by sliding it with your hands until you reach the desired position on the Bar (Fig. 10).



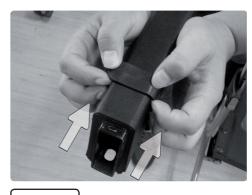


Fig. 9

Fig. 10

## 6. Assembly of the Back Bar

- 6.1 Attach the Back Bar by pressing it back into place.
- 6.2. Fix the Bar by reinserting the two screws.

# 7. Placing of the Stoppers on the Bag Hooks

Introduce the Stoppers by pressing them down onto the adaptor slot of the Bag Hooks (Fig. 11, 12).

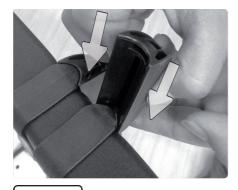



Fig. 11



Fig. 12

#### USE

- 1. Bring your rollator close to the Rollator Holder and fold it.
- 2. Tilt the rollator slightly backwards so that you can let the front wheels rest on the Rollator Holder Platform (Fig. 13).
- 3. Then, roll the rollator up to the central position of the Rollator Holder, with the extension of the rollator's one handle bar in between the minniemobil's Hook Stoppers (Fig. 14).
- 4. Use the Back Bar Belt on your minniemobil to secure the rollator (Fig. 15).
- 5. Make sure the belt is correctly tightened, and that the Stoppers are in the correct position to narrow as much as possible any movement of the fixed rollator.
- 6. To take the rollator down from the scooter, proceed the opposite way.

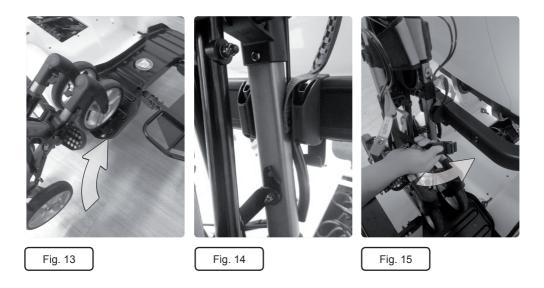

7. You may also use the hooks to hang up light weighted items (like small bags).



# Warning!

Never step onto the rollator holder or the warning triangle folded out in horizontal position.

Never drive with the triangle in horizontal position.

The maximum loading capacity of the triangle (including the rollator holder) is 15 kg, but this only applies for the very short-term purpose of stowing goods to the space under the seat.



## Warning!

Each hook has a maximum loading capacity of 5 kg / 11 lbs.



Your local dealer:				`
Date of purchase:	/	/		
Necessary for war				

#### **NOTICES TO CONSUMER**

The warranty covers all defects of the product which are demonstrably attributable to material or manufacturing defects. The warranty period is 24 months after receipt of the readiness for dispatch, at the latest after delivery. Excluded from the warranty are damages which are caused by normal wear and tear, by intent, negligent or improper operation or use. No warranty is given if you use inappropriate care products, lubricating oils and greases.



Minniemobil GmbH Grünwalder Weg 28c 82041 Oberhaching Phone: +49 89 651089 22 Info@minniemobil.com www.minniemobil.com Germany