



Safety Belt

Instructions for Assembly and Use

SAFETY BELT

The seat belt gives you a safer feeling while driving the minniemobil, and prevents you from falling off.

COMPONENTS

- 1 Male belt head
- 1 Female belt head
- 1 Left L-plate
- 1 Right L-plate
- 2 Connection screws
- 2 Connection nuts
- 4 Bottom screws
- 4 Bottom washers



ASSEMBLY (To be done by your Service Dealer)

1. Screw the L-plates right and left respectively to the bottom of the bench seat of your minniemobil, at the given spots for this purpose (Fig. 1).
2. Screw the male and female heads of the Safety Belt to the L-plates (Fig. 2).



Fig. 1



Fig. 2

USE

1. Seat yourself on the seat.
2. Pull out the Safety Belt's male head and place the belt around your abdomen (Fig. 3).
3. Insert the male head inside the female slot until you hear a characteristic "click". The belt is now secured (Fig. 4).
4. To release the Safety Belt, press the red button on the female head while holding the male head (Fig. 5 and 6).
5. The belt will roll up automatically to the initial position.



Fig. 3



Fig. 4



Fig. 5



Fig. 6



Warning!

This mobility scooter mustn't be used as a seat in or on any other vehicle or trailer, even with the safety belt fastened.

Your local dealer:

Date of purchase: / /

Necessary for warranty

NOTICES TO CONSUMER

The warranty covers all defects of the product which are demonstrably attributable to material or manufacturing defects. The warranty period is 24 months after receipt of the readiness for dispatch, at the latest after delivery. Excluded from the warranty are damages which are caused by normal wear and tear, by intent, negligent or improper operation or use. No warranty is given if you use inappropriate care products, lubricating oils and greases.



Minniemobil GmbH
Grünwalder Weg 28c
82041 Oberhaching
Phone: +49 89 651089 22
Info@minniemobil.com
www.minniemobil.com
Germany