



Seat/Back Pad Attachment and Use

SEAT/BACK PAD

The minniemobil Seat/Back Pad allows for extra comfort for your Seat and/or Backrest. Two different grades of softness are available: Soft and Extra Soft. Each pad-set contains an extra detachable bag which can be used for small items. This extra bag can only be applied for the seat use

COMPONENTS

- 1 Fabric covering
- 1 Removable pocket
- 1 Soft / Extra soft memory foam pad



REMOVABLE POCKET: PUTTING IT ON AND OFF

1. Thread the Seat/Back Pad straps into the lugs of the extra bag (Fig. 1).
2. Use the Velcro strap to fix the extra bag to the Seat/Back Pad (Fig. 2).



Fig. 1



Fig. 2

ATTACHMENT TO THE SEAT

1. Place the Seat/Back Pad on the seat with the pocket side facing the front of the seat of the minniemobil (Fig. 3).
2. Introduce the 6 straps (3 at the front and 3 at the back) of the Seat/Back Pad through the metal slots on the bottom of the seat (Fig. 4) and fasten them with their Velcro strips (Fig. 5).
3. To fix the Seat/Back Pad correctly in order to prevent it from slipping on the seat, after all the straps are fastened, tighten them firmly by pulling them again, one by one, thus readjusting the Velcro strips (Fig. 6).



Fig. 3



Fig. 4



Fig. 5



Fig. 6

ATTACHMENT TO THE BACKREST

(To be done by your Service Dealer)

1. Disassemble the shell of the Backrest by removing the four screws that fix it (Fig. 7).
2. Place the Seat/Back Pad without pocket, with the narrower side of the trapeze facing to the upper part of the Backrest (Fig. 8).
3. Introduce the 6 straps (3 at the front and 3 at the back) of the Seat/Back Pad through the metal slots on the back of the backrest and fasten them with their Velcro strips (Fig. 9).
4. To fix the Seat/Back Pad correctly in order to prevent it from slipping along the Backrest, after all the straps are fastened, tighten them firmly by pulling them again, one by one, thus readjusting the Velcro strips (Fig. 10).
5. Put the shell of the Backrest back into place.

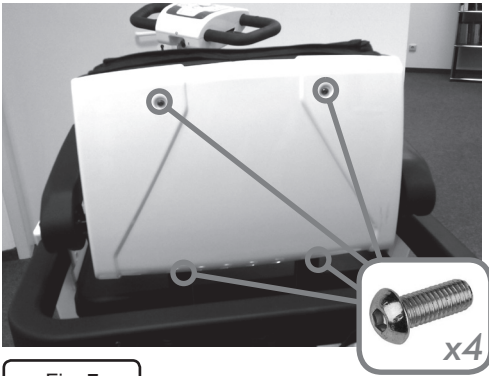


Fig. 7



Fig. 8



Fig. 9



Fig. 10

REMOVING THE MEMORY FOAM

It is possible to remove the memory foam from the fabric covering in order to replace it with an alternative material or replacement.

1. At the rear part of the Seat/Back Pad (contrary to the pocket side) there is a flap sealed with Velcro (Fig.11). Open it.
2. A zipper will be visible now. Open the zipper and take the memory foam out of the covering (Fig.12).
3. Proceed the opposite way to put the Seat/Back Pad together again.

CLEANING

We recomend to clean the seat /back pad only with a soft brush and water.



Fig. 11



Fig. 12



Warning!

Although the Seat/Back Pad is treated to be water-resistant, it may eventually get wet when exposed to rain longer than briefly.

Your local dealer:

Date of purchase: / /

Necessary for warranty

NOTICES TO CONSUMER

The warranty covers all defects of the product which are demonstrably attributable to material or manufacturing defects. The warranty period is 24 months after receipt of the readiness for dispatch, at the latest after delivery. Excluded from the warranty are damages which are caused by normal wear and tear, by intent, negligent or improper operation or use. No warranty is given if you use inappropriate care products, lubricating oils and greases.



Minniemobil GmbH
Grünwalder Weg 28c
82041 Oberhaching
Phone: +49 89 651089 22
Info@minniemobil.com
www.minniemobil.com
Germany